Rolyan* AquaForm* Corset:

A325-400 through 403

Figures referred to in the instructions are on page 6.

INDICATIONS

- To support lower back and help reduce pain
- May be used for low back pain, bulging discs, sciatica, rheumatoid arthritis, osteoporosis, and spondylolisthesis
- May be used following surgery in the lumbosacral region

CONTRAINDICATIONS

- Not for use over open wounds or infected areas

PRECAUTIONS

- **This corset is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- The corset should fit snugly and comfortably without interfering with normal breathing or circulation.
- After heating splinting material, always check its temperature before applying it to the patient.

CARE AND CLEANING

- The corset will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The corset can be cleaned with soap and lukewarm water.
- The straps and fabric front flaps can be wiped with warm water and mild soap. Allow to dry completely before reapplying the corset.

INSTRUCTIONS FOR USE

1. For patient comfort during molding, place stockinette on the area that will be covered by the corset. If softening the corset in a convection oven, use two layers of stockinette, with more over newly healed scars or skin grafts.
2. Before heating the corset, position it for proper placement. Place the corset on the back with the cutout portions at the bottom. When fitted, the top of the corset should be below the shoulder blades and the bottom should extend no more than 3/8" (1 cm) below the coccyx (A).
3. Heat the 1/8" (3.2mm) Aquaplast*-T corset as follows:

Heat pan: Heat in water at a temperature of 160° to 170°F (70°-75°C) for one minute or until transparent. Remove from heat pan at an angle to let water drain off without stretching the corset and place on a flat surface. Pat dry with a towel.

Convection oven: Heat without overlapping or folding on a baking sheet with a non-stick surface in an oven preheated to 195° to 205°F (90°-95°C) until transparent (softening may take up to 10 minutes). Remove from baking sheet and place on a flat surface.

4. Position the corset properly on the back. Bring the front flaps to the front and fasten the hook-and-loop closure on the flaps (B). Do not fasten the straps at this time.
5. While bending from the hips, the patient should lean on a table or similar supportive surface, for balance. The curvature of the spine must be maintained during the entire molding process.
6. Stroke the corset to mold it snug against the back and sides. To prevent migration of the finished corset, be especially careful to mold it to the body between the distal ribs and the iliac crest.

7. For comfort while the patient sits, flare the distal border of the corset slightly away from the body.
8. Leave the corset in place until it is completely cool. Continued stroking will help the corset maintain its shape.
9. Trim as necessary (C and D). The corset must not restrict movement of the shoulder blades and arms and it must clear the breasts. Also check for comfort while sitting. Areas that have been stitched cannot be trimmed.
10. Pull the corset open and position the **Polycushion Padding*** liner inside the corset, with paper backing toward corset. The liner might not cover the entire corset—center it before beginning to attach it. Starting from one side, slowly peel the paper from the liner and adhere the liner to the corset. Be careful to avoid wrinkling the liner. Wrinkles can be removed by lifting the liner from the corset and repositioning it.
11. If the liner extends beyond the edges of the corset, fold it over the edges (E) or trim it.
12. Reapply the corset and fasten the three D-ring straps. After the D-ring straps are threaded through the D-rings and pulled back upon themselves, the hook pieces should fasten to the straps, not to the front flaps of the corset. If a strap is too long, cut off the hook end and position one of the additional pieces that has hook on both sides onto the strap. Fasten the end of the strap to it (F).
13. Proper fit is shown in G and H.
14. If areas of pressure are apparent, or if the iliac crest is impinged when the patient sits, additional padding may be applied to those areas of the corset.

(U.S. and international patents pending.
Design copyright Smith & Nephew.)

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* Trademark of Smith & Nephew
Smith & Nephew, Inc.
Germantown, WI 53022-8205 U.S.A.
Rehabilitation Division, Smith & Nephew, Inc.
Smith & Nephew Homecraft Ltd.