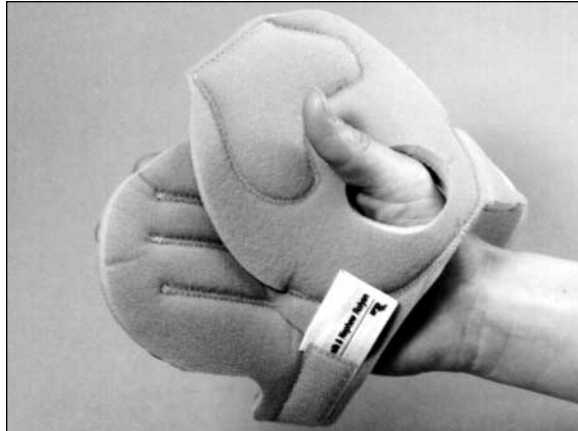
Rolyan® Palm Shield:

A812-401 and A812-402

Rolyan® Sof-Gel® Palm Shield:

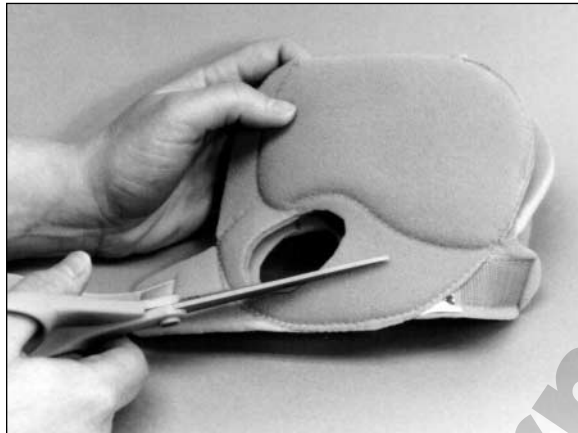
A812-405 and A812-406

A

B

C

D

INDICATIONS

- May be used for moderate to severe finger spasticity or contractures

CONTRAINDICATIONS

- Not for use if uncontrolled edema or open sores are present

PRECAUTIONS

- **The splint should fit snugly and comfortably without interfering with circulation.**
- **If unusual swelling, skin discoloration, or discomfort occurs, use should be discontinued and a healthcare professional consulted.**

CARE AND CLEANING

Fasten strap. Hand wash fabric in warm water and mild soap. Do not wring. Dry at room temperature.

Be sure the splint is completely dry before reapplying it.

INSTRUCTIONS FOR USE

1. Place the thumb through the thumb holes in both the splint and the thumb pad (A).
2. Open the fingers of the affected hand as far as possible. If the hand is extremely tight, fold the splint and slide it into the hand.
3. Line up the splint so it fits into the palm and the finger separators are in position to slide between the fingers.
4. Beginning with the index finger, slip the fingers into position on the splint with the separators upright between the fingers (B). If necessary, pull gently on the finger separators.

5. Bring the end of the strap across the hand and secure it to the hook piece on the little finger side.

6. If necessary, the material can be trimmed for proper fit. Be careful to trim only to the stitching (C).

Rolyan Sof-Gel Palm Shield

Do not trim the finger separators.

7. Check for proper positioning. The thumb should be on the outer surface of the thumb pad and the fingers should rest on the palm piece with the separators between them (D).

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