REF	Colour
091474428, 091555101	Blue
091474444, 091555093	Silver

## Introduction

The Days Foldable Steel Rollator is a multi-purpose walker manufactured from painted steel tubing with four 20 cm wheels. It incorporates a comfortable rest seat, tray, basket and locking brakes.

This walker also has adjustable handles and can be folded flat for storage or transportation.

## First Assembly

- Push the centre bar of the rear frame to the bottom of the black plastic guides.
- Fold the black locking latch on the right hand side (fig. 1A) over the bolt and tighten the black hand knob.
- Remove the plastic transit inserts from the top of the front frame.
- Insert the handle tubes into the front frame ensuring that the brake cables are not twisted and form a bow at the front of the frame.
- Position the handles at a suitable height for the user and fit the silver bolt, washer and hand knobs (fig. 1B).Tighten the hand knobs.
- The tray has 4 pips on the bottom that fit into indentations in the seat.
- The basket has two hooks at the back that clip over the tube either side of the seat.

## Operation

The Days Foldable Steel Rollator is fitted with locking brakes. To apply the brakes pull up on the brake handles (fig. 2). To lock the brakes push down on both brake handles until they click into the locked position (fig. 3). To release the brakes simply pull up on the brake handles until they click.

Before using the walker as a rest seat both brakes must be applied and the frame pushed forward to check that it is not moving. The brakes only lock the back wheels so the user must not expect the frame to be completely immobile.

If the user finds the brakes hard to apply adjust the pressure of the brake shoes on the rear wheels by using the adjuster nuts in the centre of the cables. Turn the bottom nut clockwise to decrease the pressure. Lock the adjustment by turning the locking nut until it is tight against the bottom nut. It is a case of trial and error to suit individual users' grip strength.

**WARNING!** Brakes that are adjusted to give light pressure will be less effective at preventing rear wheel movement under load.

When using the walker be aware that stability depends on all four wheels being in contact with the flat surface. Care should be taken when negotiating uneven ground, inclines, kerb etc. as stability will inevitably be reduced.

## Folding for Storage or Transportation

- Remove the locking latch from the slot in the back leg (fig. 1A).
- Pull up on the centre bar of the rear frame to fold.

## Cleaning and Maintenance


The Days Foldable Steel Rollator is designed for minimum maintenance. It is manufactured from painted steel tubing and requires the occasional wipe-over with a soft cloth. Do not use harsh abrasives. Daily, before use, check the effective operation of the brakes. Check that the wheels turn and swivel freely and there is no debris building around the axle. Periodically check that all screws, nuts and bolts are securely fastened. Check moving parts for wear.

## Specifications

Maximum user weight 130kg / 20½st; Handle height adjustment 78-96 cm; Seat height 61 cm.

## Warranty Information

All Days rollators are warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur. The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc. The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

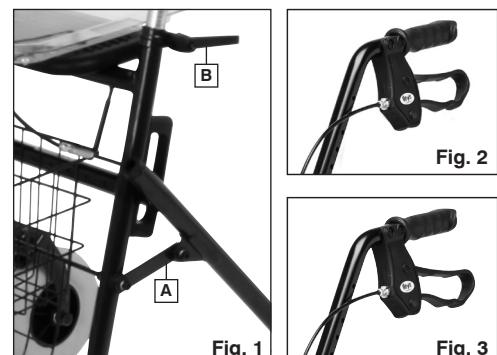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These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

 **Performance Health International Ltd.**  
Nunn Brook Road, Huthwaite,  
Sutton-in-Ashfield, Nottinghamshire,  
NG17 2HU, UK  
UK:  
Tel: 03448 730 035  
[www.performancehealth.co.uk](http://www.performancehealth.co.uk)  
International:  
Tel: +44 1623 448 706

 **Performance Health France**  
13 rue André Pingat CS 10045  
51 724 Reims Cedex, France

**Distributed in the USA by:**  
Performance Health Supply, LLC  
W68 N158 Evergreen Blvd.  
Cedarburg, WI 53012 USA  
+1 800-323-5547  
[www.performancehealth.com](http://www.performancehealth.com)

**Distributed in Australia by:**  
Ausmedic Australia, Trading as  
Performance Health ANZ  
Unit 3, 3 Basalt Road  
Pemulwuy NSW, 2145, Australia  
1300 473 422  
[www.performhealth.com.au](http://www.performhealth.com.au)

**Distribué en France par:**  
Performance Health France  
13 rue André Pingat CS 10045  
51 724 Reims Cedex, France  
00(33) 03 10 00 79 30  
[www.performancehealth.fr](http://www.performancehealth.fr)

 **EC | REP**  
CEpartner4U  
Esdoorlaan 13  
3951 DB Maarn  
The Netherlands  
[www.cepartner4u.com](http://www.cepartner4u.com)