
Rolyan® Sof•Stretch® Extension Splint:

A997-60 through 69


Rolyan® Sof•Stretch® Short Extension Splint:

A997-80 through 89


Trimline® Style: A300-10 through 14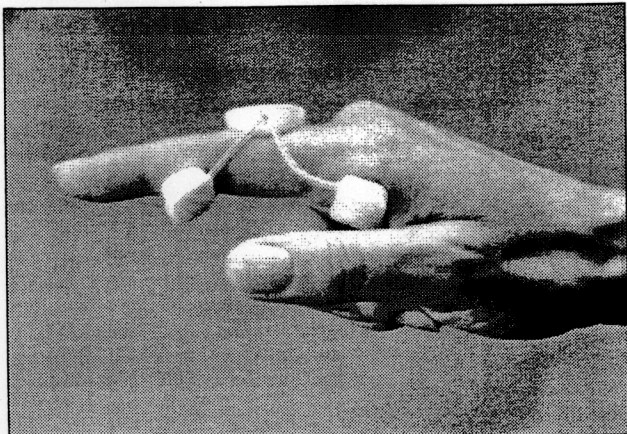


A

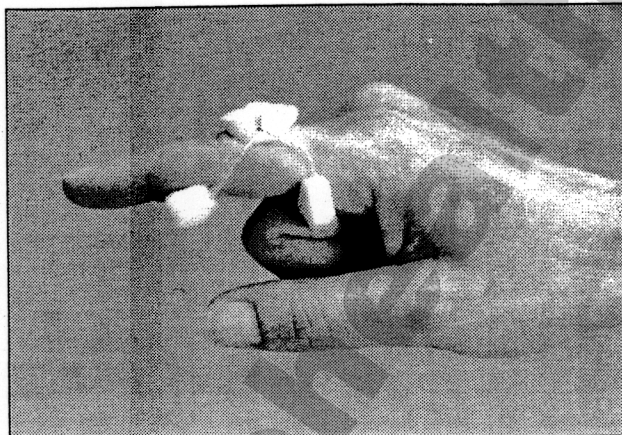B

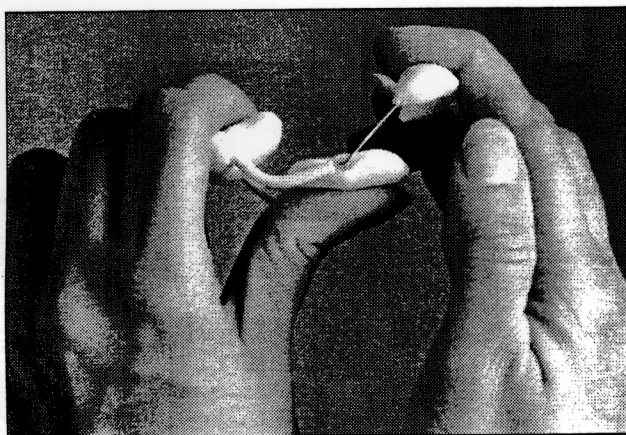C

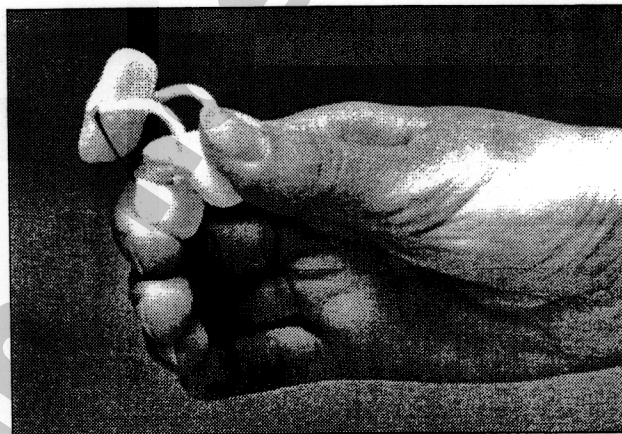D

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints listed.

INDICATIONS

- For helping correct boutonniere deformity, flexion contractures, and PIP tightness

CONTRAINDICATIONS

- Not for fixed bony contractures resulting from extrinsic tendon tightening or other structural changes
- Not for joints suffering from medial or lateral instability

PRECAUTIONS

- These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.

- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

CARE AND CLEANING

Hand wash with warm water and mild soap. Allow to air dry thoroughly before reapplication.

INSTRUCTIONS FOR USE

The Rolyan Sof•Stretch Extension Splint is shown in **A** and the Rolyan Sof•Stretch Short Extension Splint is shown in **B** (the Trimline style, which has a smaller proximal pad, should be positioned similarly).

The proximal pad has a "P" on it (except the Trimline style, which has a U-shaped proximal pad).

1. Hold the proximal and distal pads and gently slide the splint onto the finger.
 - The positioning of the proximal pad depends on which splint is being used.
 - The Rolyan Sof•Stretch Extension Splint: position the proximal pad on the volar aspect of the MCP head

—The Rolyan Sof•Stretch Short Extension Splint (both styles): position the proximal pad distal to the volar aspect of the MCP head

- The distal pad should rest at or proximal to the DIP joint, and the dorsal pad should be proximal to the PIP joint.
2. To increase the force of extension, with the splint off the finger apply pressure to the volar pads, bending them away from each other and flattening the splint (**C**).
 3. To decrease the force of extension, with the splint off the finger squeeze the two volar pads together under the dorsal pad (**D**).
 4. The width of the splint can be increased by stretching the wires laterally. Do not attempt to make major modifications in the splint—instead, use a different size.

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