Savanah® Raised Toilet Seat
AA2112, AA2114Y, AA2116

Savanah® Raised Toilet Seat With Lid
AA2112L, AA2114L, AA2116L

Instruction Sheet
70405 Issue 7

Savanah® Raised Toilet Seat

Please keep these instructions for future reference.

Intended uses

- Toilet aids allow for, or increase, independence when using the toilet, particularly by compensating for impaired movement

Instructions for installation

1. Raise the toilet seat that is attached to the toilet. The Savanah® Raised Toilet Seat should be put onto the toilet bowl.
2. Place the Savanah® Toilet Seat upside down on a flat surface and slip one of the L-shaped brackets into one of the slots in the side of the seat (**A**). Push lightly on the bracket and turn the knob clockwise until the thread engages.
Repeat with the second bracket.
3. Turn the knobs until the brackets are farther apart than the width of the toilet bowl.
4. Turn the seat over and place it on the toilet bowl with the narrow cutout at the front and the brackets near the back.
Position the seat with the front lip on the seat against the outside front of the bowl.
5. Tighten both brackets until they are snug against the bowl.
6. A properly positioned Savanah® Raised Toilet Seat is shown in (**B**).

Care and Cleaning

All parts can be cleaned with soap and hot water (up to 195°F / 90°C).
The adjusting screws can be popped out of the brackets for cleaning.

Cautions

- The toilet seat is not to be used to support more than 190kg / 420lb / 30st.
- Do not hold onto the seat as a support when getting on and off the seat.
- Be sure the toilet seat is securely in place before putting weight on it.

homecraft
Rolyan
A Patterson Company

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