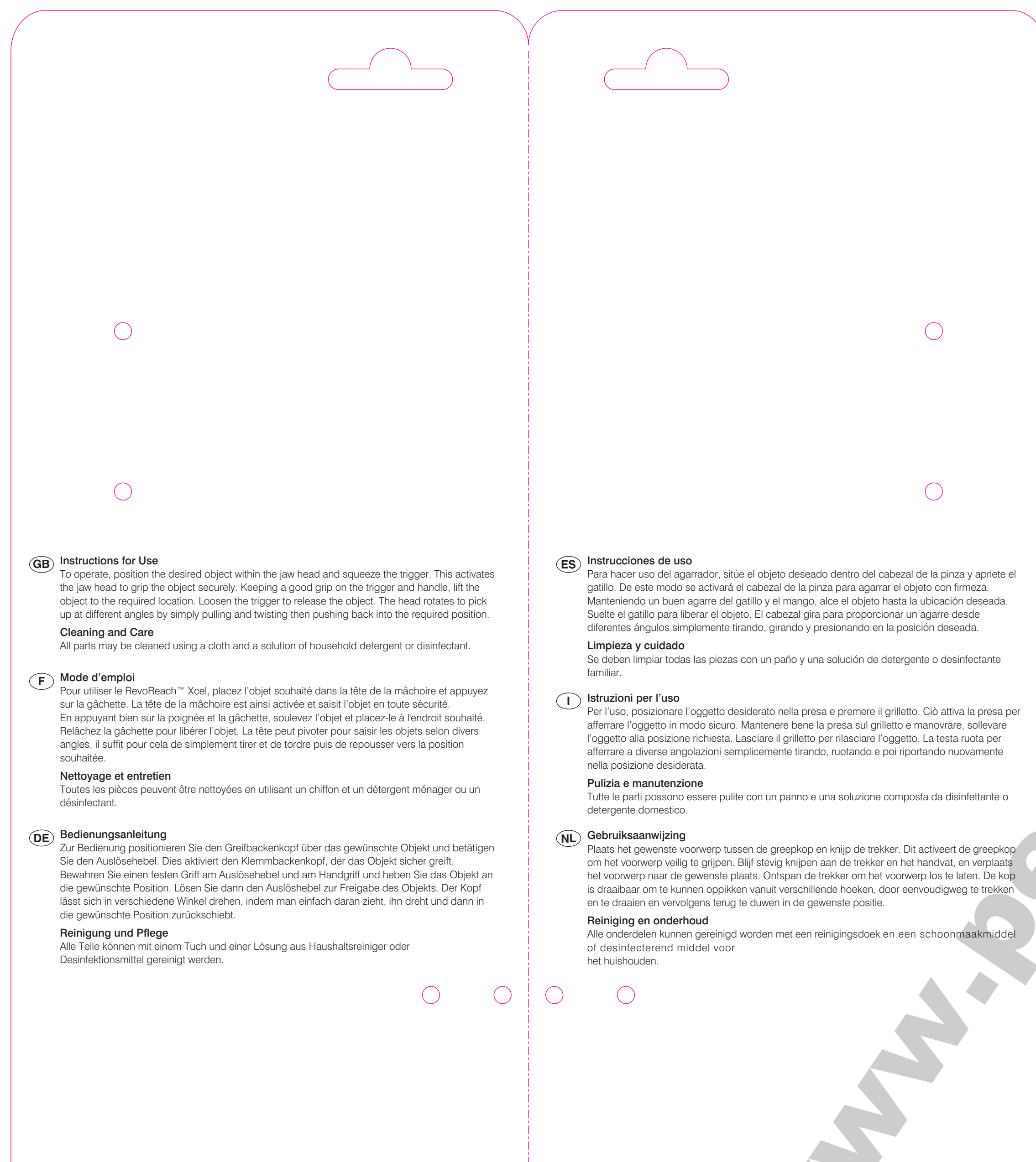
150

**(GB) Instructions for Use**

To operate, position the desired object within the jaw head and squeeze the trigger. This activates the jaw head to grip the object securely. Keeping a good grip on the trigger and handle, lift the object to the required location. Loosen the trigger to release the object. The head rotates to pick up at different angles by simply pulling and twisting then pushing back into the required position.

Cleaning and Care

All parts may be cleaned using a cloth and a solution of household detergent or disinfectant.

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150

REVOREACH
XCEL

- (GB)** The RevoReach™ Xcel is a well balanced, lightweight and robust reacher, developed to provide a secure and stable lift when picking up a wide variety of items. The ergonomically designed handle has a soft touch slip resistant trigger that allows all fingers to be used when operating, offering a more secure and comfortable grip. The 360° revolving jaw adjusts in 30° increments allowing objects to be picked up from a variety of positions.
- Ergonomic handle design reduces finger and wrist strain
 - Adjustable jaw operated by a pull and twist action offers flexibility
 - Lightweight and balanced design for ease of use and control
 - Robust construction for security and durability
 - The slip resistant jaw lining is ridged to provide additional grip when picking up objects

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09 120 4833 810mm (32")

Part No: 75345

© Patterson Medical Ltd 2010. Made in China.
Manufactured for Patterson Medical Ltd,
Huthwaite, UK, NG17 2HU.

homecraft

335

300