
Devon Support Rail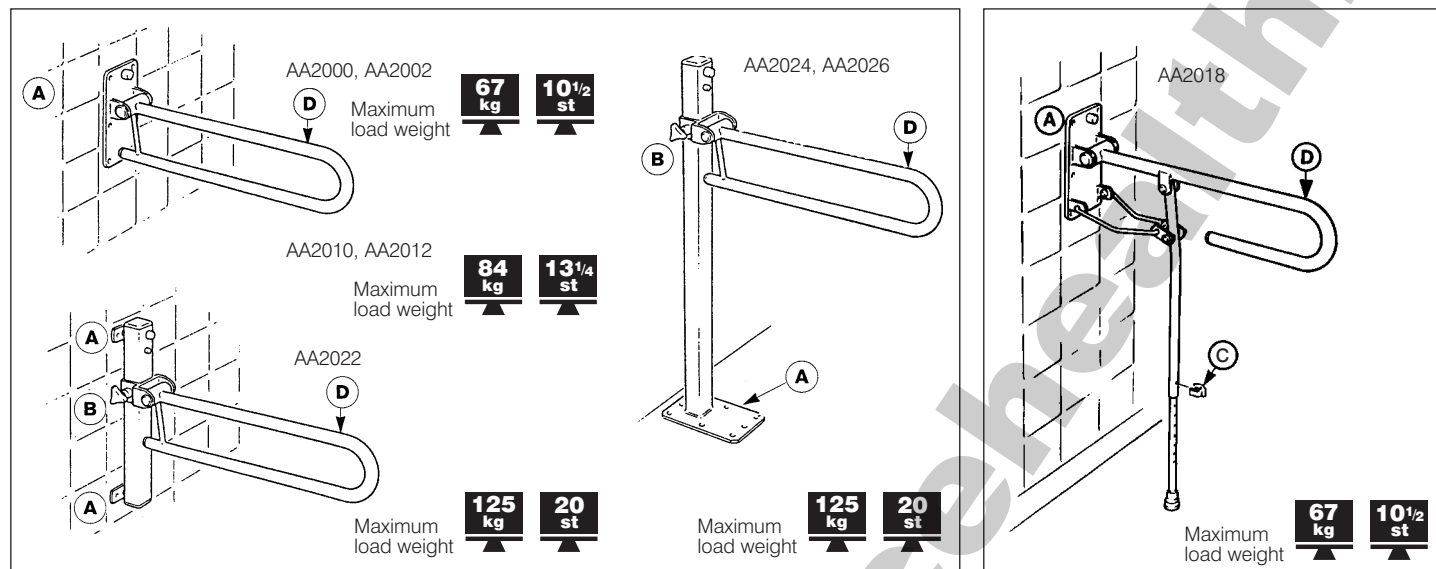

Intended uses

- To compensate for a disability by helping support a person while standing up and sitting down.

Instructions for installation

- These rails should be installed by a qualified professional, who will choose hardware appropriate for the specific installation.
- All fixing holes in the wall plate or floor plate (labelled **A**) must be used. The rail must be fastened securely so it is able to support the full weight of an average person at point **D** without the rail's bouncing.
 - Models AA2000, AA2002, AA2010, and AA2012 are not height-adjustable, and they must be permanently attached at the optimum height.
 - Model AA2022 is height-adjustable, and should be attached with the optimum height approximately in the center of the column. To adjust rail height, loosen the knob **B** and slide the rail to the desired height. Tighten the knob.
 - Models AA2024 and AA2026, which are also height-adjustable, must be mounted to the floor. To adjust rail height, loosen the knob **B** and slide the rail to the desired height. Tighten the knob.
 - Model AA2018 includes a leg for increased support. The leg length must be adjusted prior to installation, as follows:
 - Hold the wall plate against the wall and extend the rail to approximately a 45° angle with the wall.
 - Remove the pin clip (**C**) from the leg, determine the appropriate height, and replace the pin clip.
 - When determining the position of the wall plate for installation, be sure the tip of the leg is securely on the floor.
- All of the rails can be folded up out of the way when not in use.

Care and Cleaning

The rails can be cleaned with mild detergent and water.

Cautions

- Rails should be installed by a qualified professional.
- Be sure the rail is securely in place before putting weight on it.
- On height-adjustable models, be sure the height-adjustment knob is securely tightened.

Please retain these instructions for future reference.

Authorised Representative & Distributor

Homecraft Rolyan, A Patterson Medical Company,
Nunn Brook Road, Huthwaite, Sutton-in-Ashfield, Nottinghamshire, NG17 2HU, UK.
UK: Tel: 01623 448 702 Fax: 01623 557 769 www.homecraft-rolyan.com
International: Tel: +44 1623 448 706 Fax: +44 1623 448 784

Sammons Preston Rolyan, 4 Sammons Court, Bolingbrook, IL 60440-5071, USA.
Tel: 1-630-226-0000 Telefax: 1-630-226-1390

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