Savannah Slatted Bath Board:

AA1091A, B , C and D

Savannah Slatted Shower Board:

AA1092

Handle:

AA1092

Please keep these instructions for future reference.

INTENDED USES

- To compensate for a disability by providing a sitting surface for a person taking a bath or shower

CAUTIONS


- The bath or shower board is not to be used to support more than 190 kg / 420 lb. / 30 st.
- Both ends of the bath or shower board must be supported securely by the sides of the bath without an excessive overhang.
- Be sure the bath or shower is securely in place and the brackets are snug against the sides of the bath before putting weight on the board.
- Do not stand on the board.**
- Use of a slip-resistant bath mat is strongly recommended.
- Cover the bath or shower with a towel to help reduce slipperiness, if required.

INSTRUCTIONS FOR INSTALLATION

- Remove all components from the carton and snap the two brackets together as shown in **A**.
- Using the wing nuts, fasten the brackets loosely to the underside of the board and slide them as far toward the centre of the board as possible.
- Place the board on the bath, being sure of the following:
 - The board must rest securely on both sides of the bath with an overhang that does not exceed 25 mm / (1") (**B**).
 - Each end of the board must be supported by the rim of the bath at a point at least 30 mm / (1¼") in from the end of the board (**B**) (see **Specifications** for the

A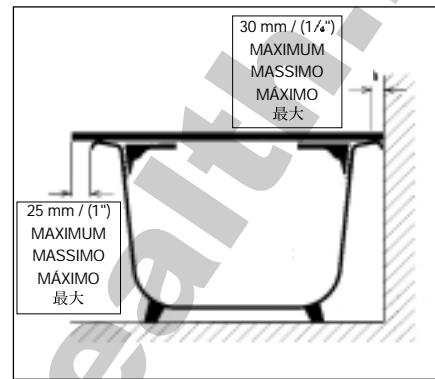

B

width suitability of each board).

- The bath must have an edge wide enough to support the ends of the board.
 - The board must be fitted squarely across the bath—**do not position it at an angle**.
 - Move the brackets toward the sides of the bath until they are snug against the sides.
 - Tighten the wing nuts securely.
 - Repeat the fitting procedure each time the bath or shower is placed on the bath.
 - The bath or shower can be used with a bath seat that enables the user to sit lower in the bath.
- Handle**
- The handle can be used to assist in transfers onto and off of the board.
- To attach the handle, position it over the holes in the butterfly-shaped cross brace on one end of the bath or shower. Fasten it into position with the screws and key provided.
 - The curve of the handle can be positioned according to personal preference.

SPECIFICATIONS

- AA1091A: 660 mm / (26") long; adjustment to suit widths inside of the bath from 483 mm / (19") to 584 mm / (23"). 232mm (9¼") wide.
- AA1091B: 711 mm / (28") long; adjustment to suit widths inside of the bath from 533 mm / (21") to 635 mm / (25"). 232mm (9¼") wide.
- AA1091C: 762 mm / (30") long; adjustment to suit widths inside of the bath from 584 mm / (23") to 686 mm / (27"). 232mm (9¼") wide.
- AA 1091D: 686 (27") long; adjustment to suit widths inside of the bath from 508mm (20") to 610mm (24"). 232mm (9¼") wide.
- AA 1093: 686mm (27") long; adjustment to suit widths inside of the bath from 508mm (20") to 610mm (24"). 356mm (14") wide.

CARE AND CLEANING

All parts can be cleaned with a household disinfectant or mild detergent and water.

NEDERLANDS

Homecraft® Savannah® Bath Board (badplank)

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Smith & Nephew Homecraft Ltd.