# Kings Modular Cutlery

### Description

This attractive and comprehensive range allows over 400 individual cutlery items to be made from 32 modular items. It caters for virtually all needs, economically and elegantly.

### Fitting and Adjustment

#### 1. Basic Fitting - Cutlery to handle (See fig 1).

To assemble a cutlery utensil to any of the handles simply push the handle firmly on to the cutlery utensil until it becomes flush with the handle. Rotate the handle to the desired position by twisting gently.

#### 2. Use of the adaptor (See fig 2).

If the angled adaptor is required, push the adaptor on to the cutlery utensil and then push the adaptor into the chosen handle. Rotate the handle to the desired position by twisting gently.

#### 3. Use of Double Ended Handles (See fig 3).

A. If using a handle with two utensils, place them into the handle, following the basic fitting instructions. If only one utensil is used then the spigot provided may be placed in the other end to close off the opening.

B. If the strap is required with a single or double piece of cutlery, then push the main cutlery item through the single hole end of the strap and place the other cutlery piece or spigot provided through the appropriate hole in the strap. Note the strap will stretch but must not be too tight. (See fig 4).

#### 4. Use of Varigrip PVC Sleeves (See fig 5).

To use the varigrip built-up sleeves 5545 etc, they must all be fitted over the 5543 slim handle.

First attach the slim handle to the cutlery utensil as in the basic fitting instructions. Then slide the chosen sleeve over the open end of the handle and push it on until it is flush with the rim of the handle. Rotate the handle to the desired position by twisting gently.

#### 5. Use of the Customised Handle Kit

One packet of FIMO is sufficient to make one handle. Work a small piece (ie 1cm block) in your hand until pliable, gradually add the rest of the block until it is all pliable. If you find this difficult the FIMO can be placed on a radiator for 1 - 2 minutes to soften.

Form the shape required for the individuals hand and push the pinched end of the metal inner core into the FIMO leaving the rim of the open end protruding slightly.

Take care not to get any material in the open end and check that the moulded shape has not been distorted.

Harden in a standard oven at a maximum temperature of 130°C (265°F) for 20 - 30 minutes. Note: do not fit the cutlery utensil until it has been hardened, and is cool.

#### To Disassemble

If you wish to change cutlery utensils or handles simply twist the handle gently and pull out.

#### Cleaning and Care

All items in the range, including the FIMO can be washed either in the usual manner or in a dishwasher.

Manufactured for Homecraft AbilityOne.

Made in China.

Fig. 1

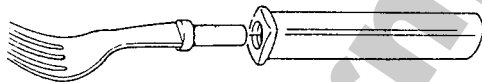
Fig. 2

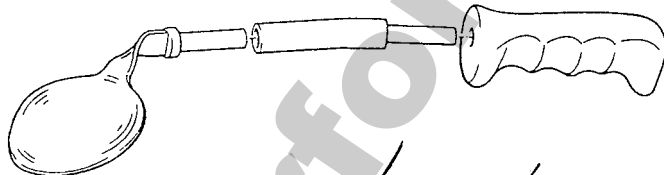
Fig. 3

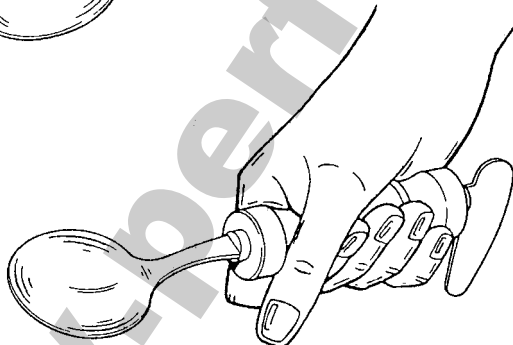
Fig. 4

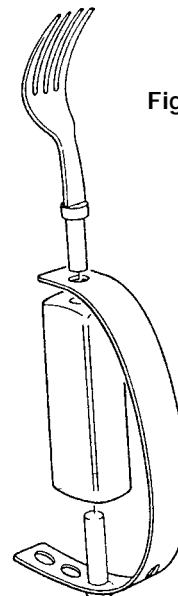
Fig. 5

# Homecraft AbilityOne

Homecraft AbilityOne

Shelley Close, Lowmoor Business Park,  
Kirkby-in-Ashfield,

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