09 109 2097 AA4200 Adjustable Padded Leg Rest Instruction Sheet

Please read these instructions carefully and retain for future reference

Safety

Do not sit or stand on the Leg Rest, the product is not intended to support full body weight.

Installation and Adjustment

These products are supplied flat-packed and are simple to assemble or dis-assemble again without tools, using the instructions opposite:

1. Adjust the legs attached to the padded seat to an angle of 60° by releasing the triangular knob.
2. Remove the two pin clips from the base legs.
3. Slide the leg assembly over the legs attached to the seat. Adjust the legs to the required position and clip through the holes with the pins provided.

4. Angle the padded top to the desired angle and fasten the triangular knob to lock in position.
5. To disassemble, reverse the procedure.

Cleaning and Care

All metal parts may be cleaned using a brush or a cloth, and a solution of mild detergent and disinfectant (e.g. Detox).

Manufactured for: Patterson Medical Ltd, Nunn Brook Road, Huthwaite, Sutton-in-Ashfield, Nottinghamshire, NG17 2HU, UK.

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