Brevetti Sock and Stocking Aid

Batty Sock and Stocking Aid

AA4666 (09 109 5603)

AA4664 (09 109 5587)

70004 IS7

Please keep these instructions for future reference.

Intended Uses

- To compensate for a disability by helping a person put on socks or stockings without unnecessary bending at the waist or hips

Assembly

The Brevetti Sock and Stocking Aid (AA4666) comes fully assembled.

The Batty Sock and Stocking Aid (AA4664) consists of the following components:

- collar (labelled A in the drawing)
- angled section (B)
- middle section (C) with two locking levers (F)
- handle (D)
- shoehorn (E)

Assemble the Batty as follows:

- Place the collar (A) on a flat surface with the rim down.
- Put the spindle end of the angled section (B) into either of the slotted holes on the collar, positioning it for right- or left-hand use (assembly for right-hand use is illustrated).
- Push the angled section into the hole completely, and then rotate it until it locks in place.
- Push the middle section (C) onto the angled section and rotate the locking lever to the locked position (G). (H shows unlocked position.)
- Fit the handle (D) onto the middle section and lock it in position.
- The shoehorn (E) can be attached to the middle section (C) by first detaching the handle or angled section and then fitting the shoehorn onto the middle section.

Instructions for Use

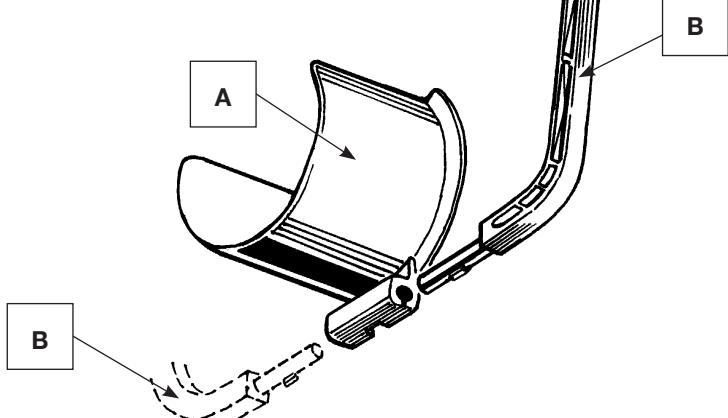
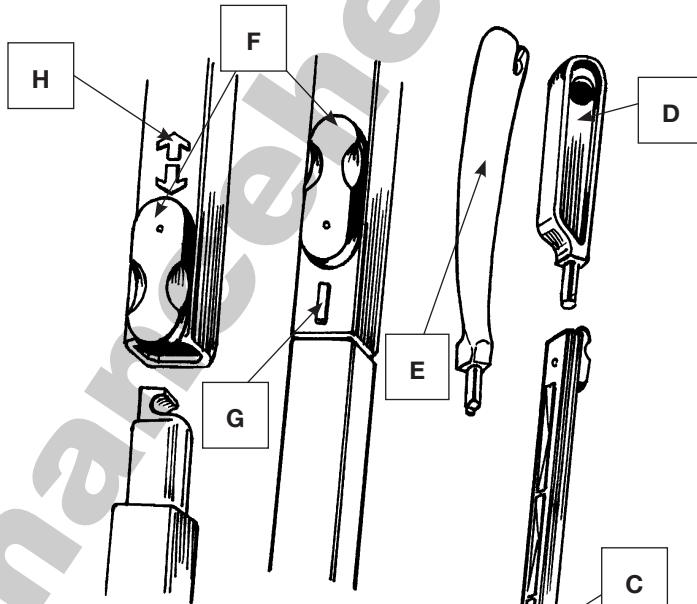
Brevetti and Batty Sock and Stocking Aids

See pages 4, 5 and 6.

Care and Cleaning

The sock aid can be cleaned with mild detergent and water.

Batty Sock and Stocking Aid-Components
Batty hulpmiddel voor aantrekken van sokken en kousen-onderdelen
Enfile-bas et chaussettes Batty-Composants
Batty Socken- und Strumpfanzieher-Komponenten
Infilacalze/calzini Batty-Componenti
Componentes del calzador para medias y calcetines Batty

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1

ENGLISH

1. Lay the aid on a flat surface with the rim of the collar down. With the garment right-side out, hold it with the heel facing into the curve of the collar.

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2

ENGLISH

2. Stretch the top of the garment over the collar and rub along it to pull it down onto the outside of the collar toward the rim.

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3

ENGLISH

3. Continue rubbing and pulling the sock down until the heel is completely on the outside of the collar and the toe is inside the collar. Be sure that the top of the garment does not extend past the rim of the sock aid.

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4

ENGLISH

- Lower the sock aid to the floor in front of the foot. Grasp the handle and slip the sock aid over the foot until the toes are in the toe of the garment. Wiggling the toes may help the garment move onto the foot. Pull the sock aid gently onto the foot.

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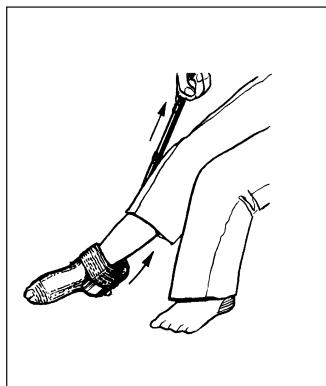
5

ENGLISH

- Lift the heel from the floor, straighten the knee, and point the toes. Pull gently on the handle to unroll the garment and fit it smoothly over the foot.

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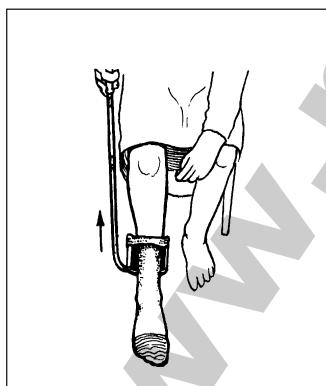
6

ENGLISH

- Continue to pull the sock aid up the leg until it releases from the garment.

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7

ENGLISH

7. The slot in the shoehorn can be used to help remove the garment.

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Please retain these instructions for future reference.

Manufactured for Patterson Medical Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK: Tel: 08444 124 330 Fax: 08448 730 100

www.pattersonmedical.co.uk

International: Tel: +44 1623 448 706

Fax: +44 1623 448 784

Patterson Medical Products, Inc.

1000 Remington Boulevard, Suite 210,
Bolingbrook, IL 60440-5071, USA.
Tel: 1-800-323-5547 Telefax: 1-800-547-4333

Patterson Medical Australia

PO Box 1006, North Ryde, NSW Australia 2113.
Tel: 1300 473 422 Fax: 1300 766 473

Patterson Medical France

Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 526 115 Fax: 03 24 529 034

PATTERSON
MEDICAL